Easy Expression™ Bustier

Thank you for choosing Easy Expression Bustier!

Why Easy Expression™ Bustier?

Easy Expression Bustier is the best solution, if you want to do other things while pumping, e.g. reading or surfing the internet. The comfortable Easy Expression Bustier enables you to pump hands-free. Easy Expression Bustier is made of a soft cotton/elastane blend, is easy to wear and available in different sizes.

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Instructions

1 Dress bustier, leave zipper a little bit open.

2 Place breastshield under bustier so that your nipple is centred in the tunnel of the breastshield. Then close the zipper.

3 Attach bottle with connector to the breastshield. Now pump handsfree.

For a smooth and secure pumping session with Easy Expression Bustier Medela recommends sitting upright.

Wash instructions

- I Wash with similar colour only.
- I Machine wash, max temperature 30 °C.
- I Only non-chlorine bleach when needed.
- I Tumble dry at lowest temperature only.
- I No iron.

Material

89% Cotton, 11% Spandex

Visit our website www.medela.com there you can find a detailed description of the correct use of Easy Expression and more information about Medela products.

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